



DRUM WHEEL BRAKE KIT

QUICK INSTALLATION GUIDE

This guide is intended to explain the correct set up procedure to install the drum wheel brake kit for compatible Aspire Wheelchairs.

For further operating instructions, refer to the product User Manual.

CODE	DESCRIPTION	COMPATIBLE WITH					
		EVOKE 2	EVOKE 2 JNR	REHAB RX JNR	REHAB RX	REHAB RS	ASSIST 2
WCA848510	Rear Wheels w/ Drum Brakes Kit – 16in	—	●	●	●	●	—
WCA848520	Rear Wheels w/ Drum Brakes Kit – 20in	—	●	○	●	●	—
WCA848500	Rear Wheels w/ Drum Brakes Kit – 24in	●	—	—	○	○	●

○ – Comes as Standard Feature

Tools Required



24mm Spanner



10mm Spanner



4mm Allen Key

Carton Contents



2x

Long Silver Bolts w/ Offset Receiver



2x

Small Black Nuts




2x

Small Silver Spacers




2x

Large Silver Spacers

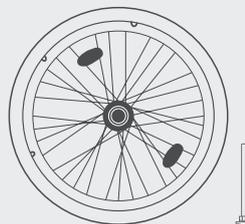

2x

Sets of Rear Wheels & Quick Release Axles

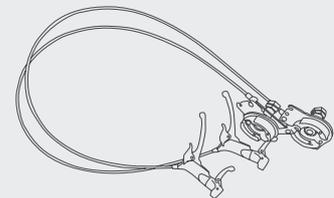

2x

Sets of Drum Brakes w/ Handles and Brakes Attached



8x

Large Silver Nuts





2x

Small Black Bolts



INSTALLATION GUIDE

Removing Camber Kit

- 1 Remove quick release axle from wheelchair by pushing in spring button and pulling wheel outwards.
- 2 Lay the wheelchair on its side, remove camber kit from wheelchair frame using a 24mm spanner.

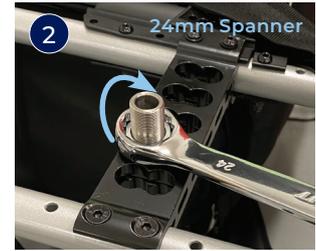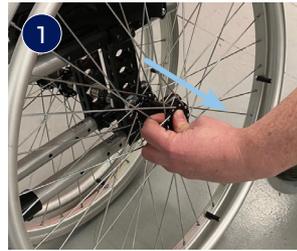

Installing Drum Brakes

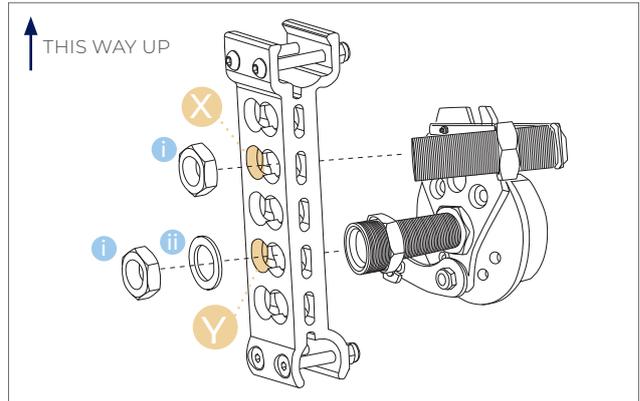
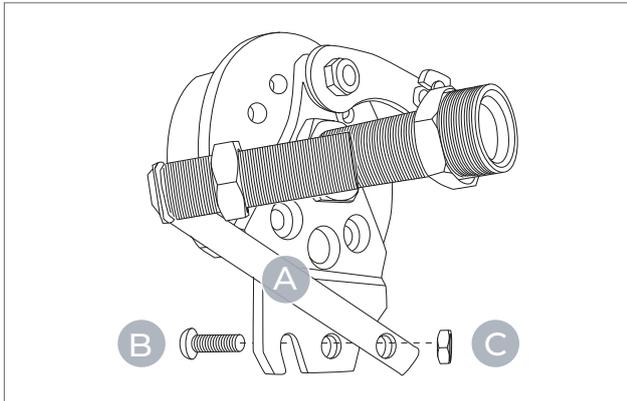


- 3 Install long silver bolt **A** onto brake kit and secure with small black nut and bolt **B**, **C** with 4mm allen key and 10mm spanner
- 4 Ensure the direction of the brake kit is facing upwards towards the top of the wheelchair.

- 5 Slide drum brake kit bolts through holes **X** and **Y**
- 6 Stand wheelchair back onto it's wheels (from side lying position) ensuring cable pin stays pointing towards the top of the wheelchair.
- 7 Mount hardware **i** **ii** on bolts on drum brake kit and drum brake. Secure in place with 24mm spanner.

Installing Hand Brakes

- 1 Mount hand brake to wheelchair handle & fasten in place with a 4mm Allen key.
- 2 Fasten wheel to wheelchair by locking in place with the quick release axle provided in the box.
- 3 Repeat steps 1 to 8 for the other side.

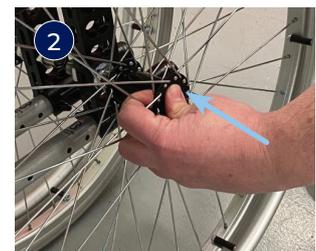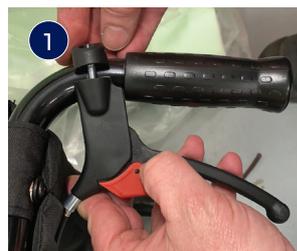

Adjusting Brake Strength

If brakes require tightening or loosening, follow below steps.

- 1 Remove wheel and quick release axle from wheelchair, slide out brake cable.
- 2 Adjust brake strength: Tighten (Clockwise) & Loosen (Anti-Clockwise).
- 3 Reposition brake cable & wheel and lock into place with quick release axle.



ASUM58.26 170424